



MeMo System was developed in response to a serious foot dysfunction in children of growing age. The idea of the **MeMo** System is early detection and prophylactic, as well as therapeutic approach to foot diseases that, if left untreated, lead to spine and posture dysfunctions.

Control and diagnostic area is its outsole and heel. They are divided into six numbered zones with cambered stripes. When wearing **MeMo** shoes, it is possible to observe the degree of stripes' abrasion. Depending on worn out zone, quick and precise diagnosis can be determined and corrective insoles can be



Unique diagnostic sole indicating foot dysfunction level

Zones 5 and 6

Zones of serious dysfunction

Abrasion in 5th zone of the out sole and 6th zone of the heel indicate the necessity of appointment with GP.

Zone 6

Very serious dysfunction

Abrasion in the 6th zone indicates the need of appointment with GP.

Zone 1

Normal zone

Any abrasion in this area means **HEALTHY FEET. CONGRATULATIONS !!!**
In this situation Memo shoes act preventively - use beige insole



Zones 3 and 4

Zones of dysfunction

Indicate case of foot and/or knee defect. It is necessary to use the orange Memo corrective insole and to have an appointment with GP.

Zone 3

Dysfunction zone

Indicates foot and/or knee defect. It is necessary to apply the orange Memo corrective insole while wearing Memo footwear all time. If abnormality persists, you should have an appointment with GP.

Zones 2

Risk zone

Danger of a defect or illness. Abrasion of the outsole in this zone suggests the necessity of using grey insole and permanent usage of Memo shoes. If during the period of using grey insole the outsole abrasion does not return to normal zone, it is necessary to have an appointment with GP.

PURPOSES OF MEMO SYSTEM

1. Foot stabilization during the growth period.
2. Providing proper body posture and teaching proper walk.
3. Controlling proper foot growth.
4. Eliminating majority of foot and knee joint dysfunctions.
5. Limiting costs of medical care.

SET OF CORRECTIVE INSERTS

Prevent poor posture and body alignment, bone deformity, and disorder/dysfunction of balance and coordination caused by foot dysfunction.



MEMO SYSTEM WORKS

Diagnostic - because it contains zoning system (on the sole) that makes possible for parents to get first diagnosis.

Prophylactic - because special shoe design protects feet and prevents eventual dysfunction, as well as provides the proper posture.

Therapeutic - because, after applying particular insert, it corrects posture dysfunctions.